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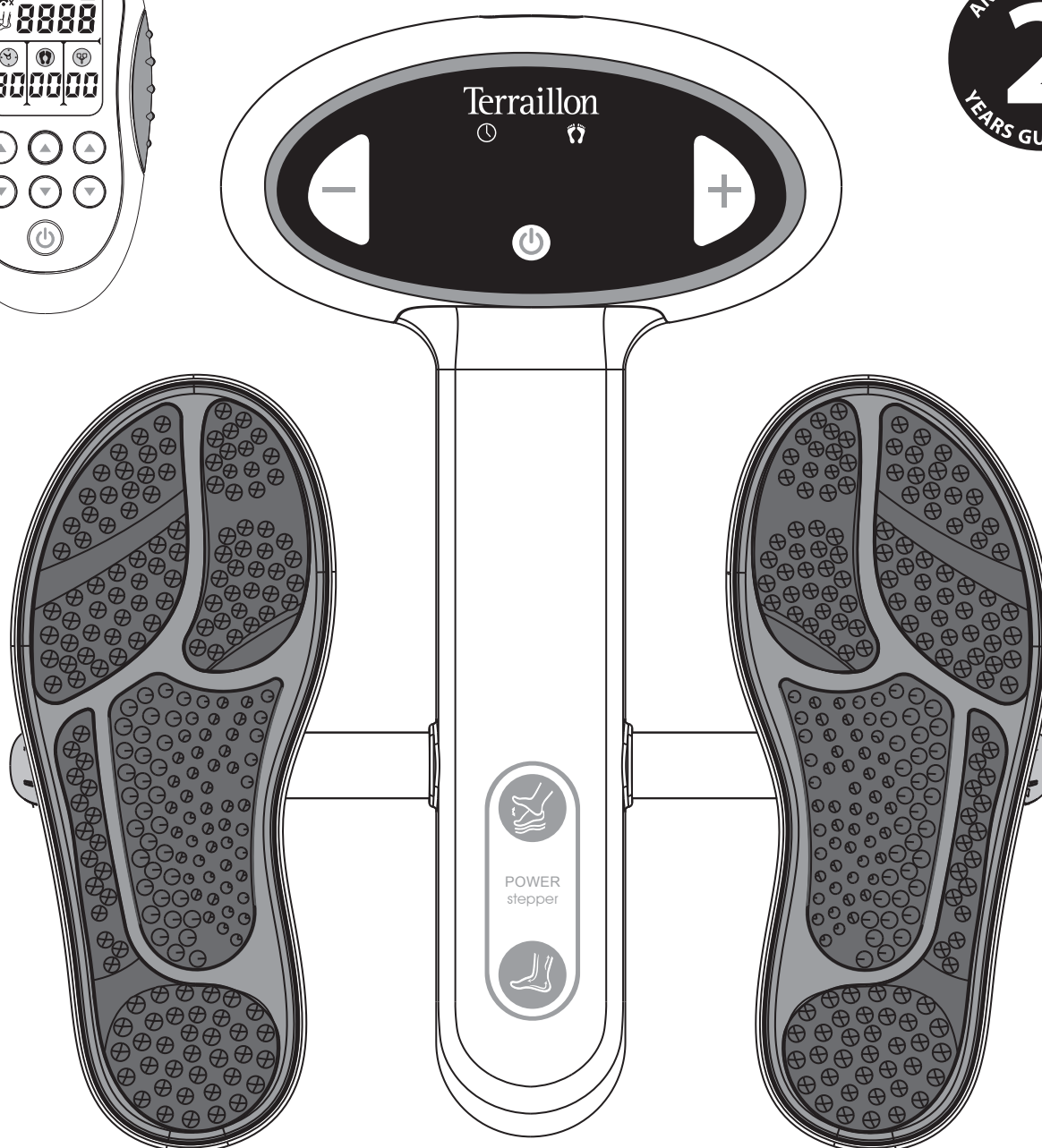
## POWER STEPPER - INSTRUCTION MANUAL

*Dear Customer,*

*Thank you for purchasing this product. Please read these instructions carefully to take full advantage of your product.*

### INTRODUCTION

POWER STEPPER by Terraillon is a device combining EMS and TENS, which can help improve blood circulation effectively. And pain relief EMS stands for Electrical Muscle Stimulation, is a non-invasive muscle exercise and muscle toning technique that is a very useful supplement to your workout routine. EMS machine operates by repetitively applying an adjustable intensity and pacing frequency with contraction and trigger intervals.

EMS has been widely used in hospitals and sports clinics for improving blood circulation, reducing swollen feet and ankles, alleviating tired and aching of legs and provide ankle dorsiflexion in individuals who have foot drop following an upper motor neuron injury or disease TENS stands for Transcutaneous Electrical Nerve Stimulation. This technology is clinically proven to be a safe and effective solution for relieving even persistent pain. It is 100% drug-free therapy that is approved and has been used for more than 30 years by physicians and pain clinics around the world.

### EXPLANATION OF THE POWER STEPPER

Poor circulation may cause tired, aching and heavy legs. Most people get tired, aching legs and swelling feet some time in their lives-usually caused by standing or sitting for a long time. The Power Stepper uses Electrical Muscle Stimulator (EMS) applied primarily through two large foot pads, sending electrical pulses to plantar muscle. The physiological result is that the foot and calf muscles contract and release, creating a pumping action. This action imitates the body's natural circulatory system of legs while walking. As the calf muscle contracts, the venous return is facilitated, avoiding stagnation of the blood and the feeling of heavy legs.

### REASONS FOR USE

- Reduce swollen feet & ankles
- Alleviate tired & aching legs
- Improve blood circulation
- Ease joint & muscle pain

### SAFETY INFORMATION

#### CONTRAINDICATIONS

- Do not use this device on patients who have a cardiac pacemaker, implanted defibrillator, or other implanted metallic or electronic device.
- Suffer from Existing Deep Vein Thrombosis ("DVT"), in case of doubt, consult your doctor beforehand
- If you are pregnant.

#### PRECAUTIONS

- 1) Some patients may experience skin irritation or hypersensitivity due to the electrical stimulation or rubber electrode;
- 2) Patients with suspected or diagnosed heart disease must obligatorily follow precautions recommended by their physicians; and
- 3) Patients with suspected or diagnosed epilepsy must obligatorily follow precautions recommended by their physicians.
- 4) Use caution when the patient has a tendency to bleed internally, such as following an injury or fracture. Ask your doctor for advice.
- 5) Use caution following recent surgical procedures when stimulation may disrupt the patient's healing process.
- 6) Keep this device out of the reach of children;
- 7) Do not modify or disassemble the device yourself.
- 8) Store the device in the dry, clean place.
- 9) Use this device only with electrode and accessories recommended by the manufacturer.

## POWER STEPPER - INSTRUCTION MANUAL

### WARNINGS

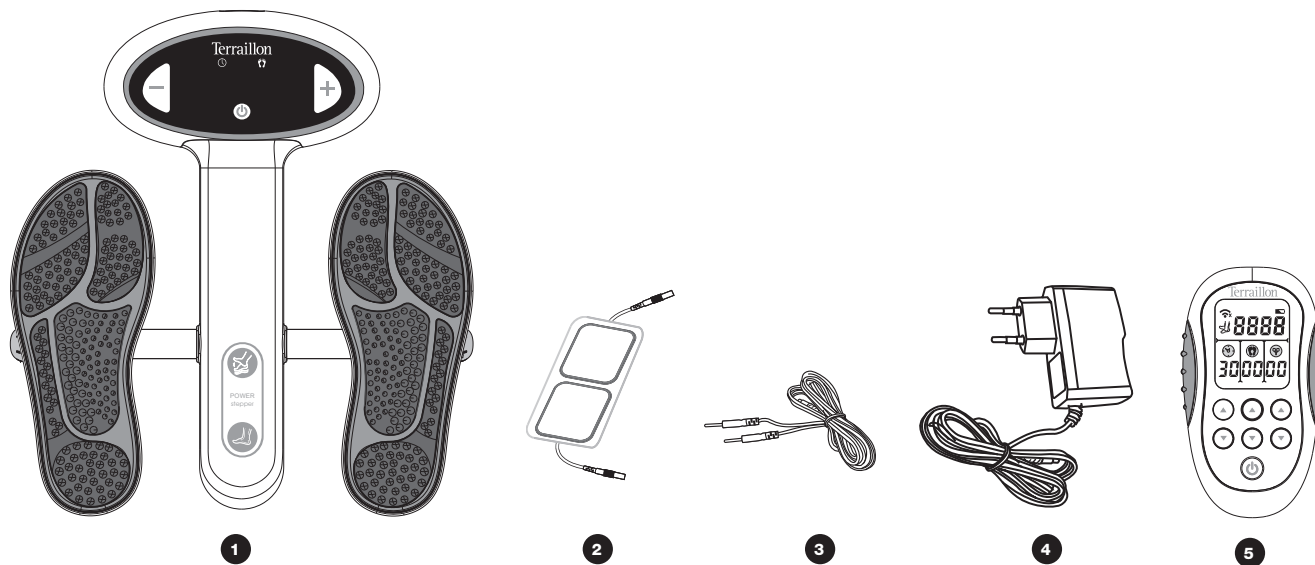
- 1) Do not apply stimulation over open wounds or rashes, or over swollen, red, infected, or inflamed areas or skin eruptions (e.g., phlebitis, thrombophlebitis, varicose veins);
- 2) Do not apply stimulation over, or in proximity to, cancerous lesions;
- 3) Do not apply stimulation in the presence of electronic monitoring equipment (e.g., cardiac monitors, ECG alarms), which may not operate properly when the electrical stimulation device is in use;
- 4) Do not apply stimulation when is in the bath or in the shower;
- 5) Do not apply stimulation during sleep; and
- 6) Do not apply stimulation while the patient is driving, operating machinery, or during any activity in which electrical stimulation can put the patient at risk of injury.
- 7) Consult with your doctor before using this device, because the device may cause cardiac rhythm in susceptible individuals; and
- 8) Apply stimulation only to normal, intact, clean, healthy skin.
- 9) Patients with arterial or venous thrombosis or thrombophlebitis are at risk of developing embolisms when electrical stimulation is applied over or adjacent to the vessels containing the thrombus. If a patient has a history of deep vein thrombosis, even many years past, the affected area have to not be stimulated. Seek your doctor.
- 10) Fresh fractures should not be stimulated in order to avoid unwanted motion.
- 11) Stimulation should not be applied immediately following trauma or to tissues susceptible to hemorrhage.
- 12) Do not apply electrodes directly over the eyes or inside body cavities.
- 13) Do not operate in close proximity to shortwave or microwave therapy equipment
- 14) Do not apply stimulation over the patient's head or neck because this could cause severe muscle spasms resulting in closure of the airway, difficulty in breathing, or adverse effects on heart rhythm or blood pressure;
- 15) Do not apply stimulation across the patient's chest, because the introduction of electrical current into the chest may cause rhythm disturbances to the patient's heart.
- 16) In case of adverse reactions (irritations/inflammation/burns) consult your doctor immediately.
- 17) Simultaneous connection of a PATIENT to a high frequency surgical device may result in burns at the site of the STIMULATOR electrodes and possible damage to the STIMULATOR.
- 18) Application of electrodes near the thorax may increase the risk of cardiac fibrillation.
- 19) Stimulation should not be applied across or through the head, directly on the eyes, covering the mouth, on the front of the neck, (especially the carotid sinus), or from electrodes placed on the chest and the upper back or crossing over the heart.
- 20) It is recommended not to exceed a current density of 2 mA /cm<sup>2</sup>, otherwise skin irritations or burns can occur. We recommend that:
  - Please use specific size electrodes provided by manufacturer to avoid skin irritations or burns.
  - For smaller electrodes, the maximum current setting of the waveform should be appropriately reduced.
  - Please place the electrodes carefully, ensure that the entire surface of the electrode has good contact with the skin.

### CAUTIONS

- 1) Keep yourself informed of the contraindications.
- 2) Read, understand, and practice the warnings, cautions and operating instructions. Know the limitations and hazards associated with using any device. Always follow the operating instructions prescribed by your healthcare practitioner.
- 3) DO NOT operate this unit in an environment where other devices are being used that intentionally radiates electromagnetic energy in an unshielded manner.
- 4) DO NOT use sharp objects such as a pencil point or ballpoint pen to operate the buttons on the control panel.
- 5) This device should not be used adjacent to or stacked with other equipment.
- 6) This device needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in the manual.
- 7) Do not use other RF equipment near the technical booklet.
- 8) This device has been thoroughly test and inspected to assure proper performance and operation!
- 9) Do not use RFID systems near the device.
- 10) Before administering any treatment to a patient you should become acquainted with the operating procedures for each mode of treatment available, as well as the indications, contraindications, warnings and precautions.


POWER STEPPER - INSTRUCTION MANUAL

COMPOSITION


- 1 Power Stepper
- 2 Electrode pads (40 mm x 40 mm)
- 3 Cable
- 4 Adaptor
- 5 Remote controller + batteries included (3 x AAA)

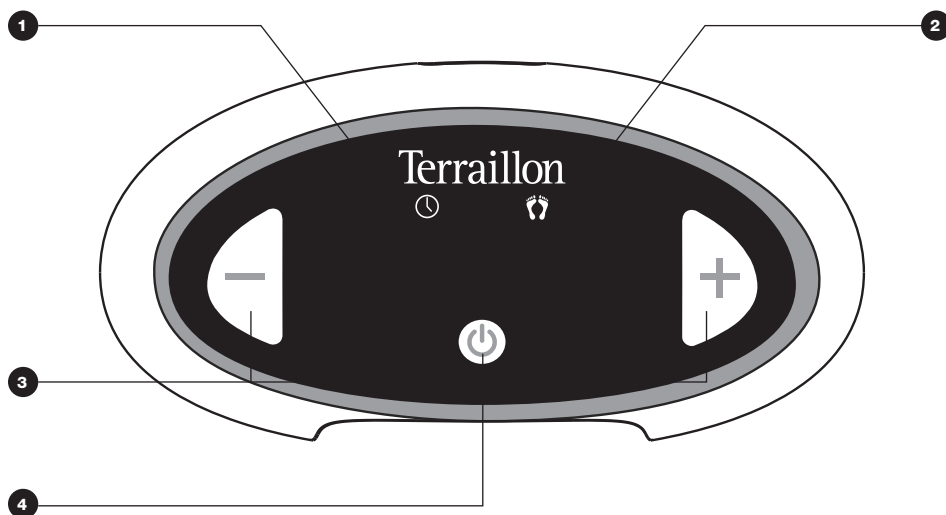PRESENTATION


- 1 Display treatment time
- 2 Display intensity of foot stimulation
- 3 Increase or decrease the intensity of foot stimulation
- 4 Main device power button

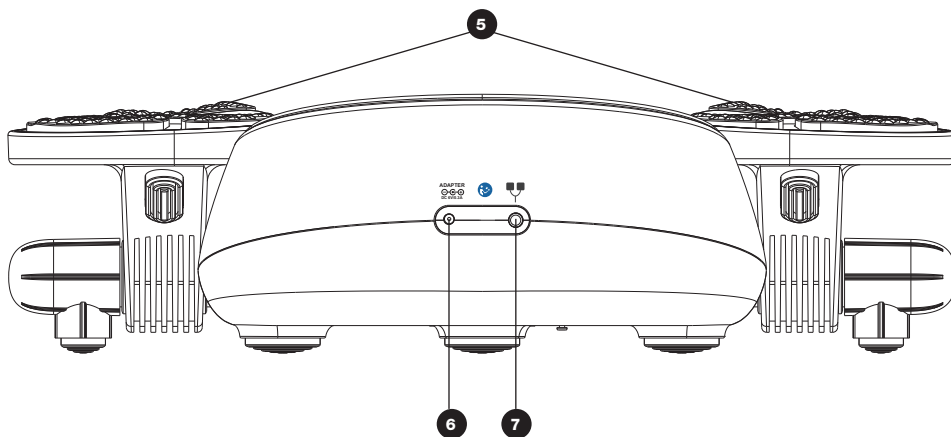POWER STEPPER - INSTRUCTION MANUAL

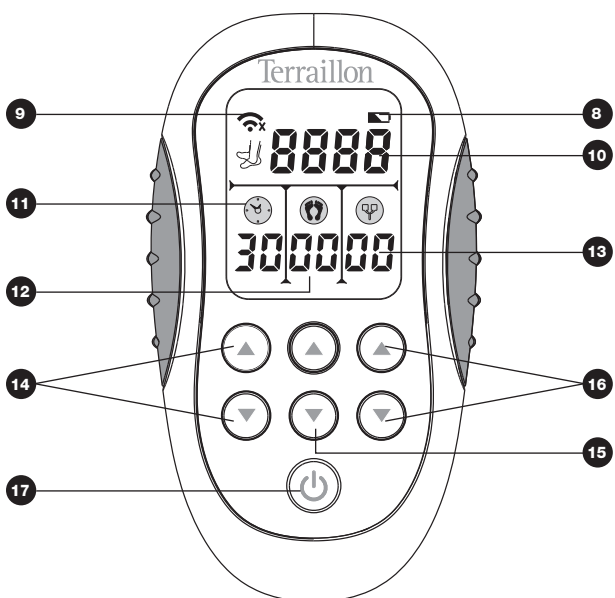- 5 Foot pads
- 6 Adapter socket
- 7 Output socket for body stimulation



- 8 Low battery icon
- 9 Poor wireless connection problem icon
- 10 Display steps
- 11 Display treatment time
- 12 Display intensity of foot stimulation
- 13 Display intensity of body stimulation
- 14 Increase or decrease the treatment time
- 15 Increase or decrease the Intensity of foot stimulation
- 16 Increase or decrease the Intensity of body stimulation
- 17 Remote control power button

## POWER STEPPER - INSTRUCTION MANUAL
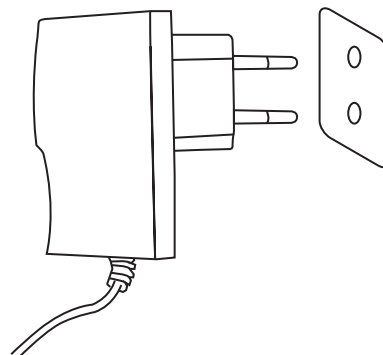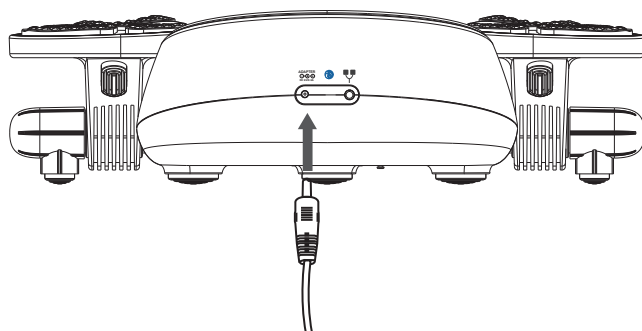
## INSTRUCTION FOR USE

## STEPS TO CONNECT THE ADAPTOR

- Insert the adaptor tip into the rear socket of the device. (Hold the main unit firmly when inserting the tip into the adapter socket).
- Insert the adaptor into the nearest power supply socket. Make sure the power voltage is appropriate.


## CAUTION:

- Please use the original adaptor. The user cannot assemble or change the specification of the adaptor. Personal injury or damage to the unit may occur if you do not follow above instructions.
- Always place the device Power Stepper on the floor where is easy unplug it at any time.

## CHECK/REPLACE THE BATTERY (NOT INCLUDED)

For the convenience, the device is designed with battery power supply.

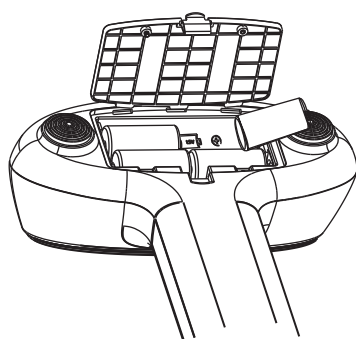- Open the battery compartment cover.
- Insert 4 batteries (type C / LR14) into the battery compartment, respecting the polarities (the polarities are indicated inside the battery compartment).
- Close the battery compartment cover.
- Do the same with remote control (3 x AAA included).



## CAUTION:



- When connecting from the mains plug, the device will cut off the internal batteries supply automatically.
- Remove the batteries if the device does not work for a long time.
- Do not mix old and new batteries or different types of batteries.
- Warning: If batteries leak and touch your skin or get into your eyes, wash with large amounts of water immediately.
- Keep batteries out of reach of children.
- Do not use rechargeable batteries. Remove exhausted batteries from the unit.

## POWER STEPPER - INSTRUCTION MANUAL

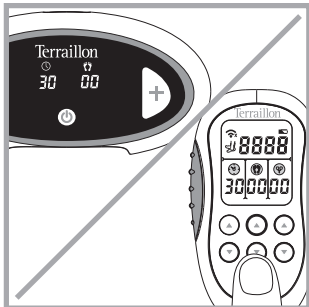
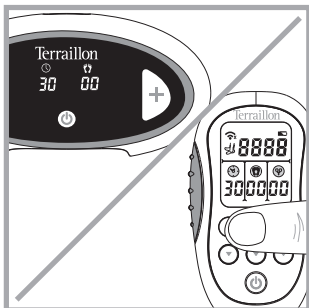
## FOOT AND LEG STIMULATION (👣)

	<p><b>Step 1 - Place the device on the floor.</b></p> <ul style="list-style-type: none"> <li>Place the device on the floor a comfortable distance from your chair. Sit down on your chair.</li> </ul> <p><b>WARNING:</b> Never stand on the Power Stepper.</p>
	<p><b>Step 2 - Place both bare feet onto the footpads.</b></p> <ul style="list-style-type: none"> <li>Remove all socks and stockings and place bare feet onto the footpads. The arch of your foot should be positioned on the raised instep on the footpads. It does not matter if your feet are smaller or bigger than the footpads.</li> <li>Both the footboards of the POWER STEPPER can tilt back and forth improve create ankle joint movement for increasing circulation and reducing swelling. The footboards will tilt back and forth as per your feet rocking motion.</li> <li>This type of exercise therapy is recommended widely by health professionals to improve circulation and, when used in conjunction stimulation, the results are significant.</li> </ul> <p><b>CAUTION:</b> Two feet are required to create the stimulation at the desired level. With only one foot, the device does not go beyond level 10 (safety level). Sit with your knees both at a 90 degree angle.</p>
	<p><b>Step 3 - Turn on the device</b></p> <ul style="list-style-type: none"> <li>Press the power button [🔌] on the device or use the remote control.</li> </ul> <p>In the default setting.</p> <ul style="list-style-type: none"> <li>The screen displays [🕒] 30 minutes, [👣] 00 output intensity, [📶] 00 output intensity. [👣] display 0000.</li> </ul>
	<p><b>Step 4- Setting the treatment time</b></p> <ul style="list-style-type: none"> <li>Set the treatment time by using the ▲/▼ [🕒] buttons on the remote control.</li> <li>The treatment time can be adjusted from 5 to 90 minutes.</li> </ul>

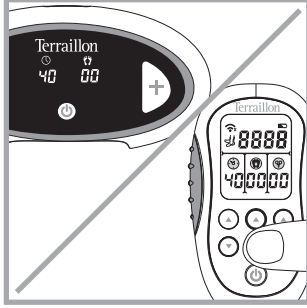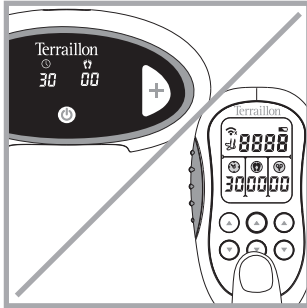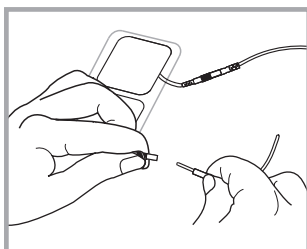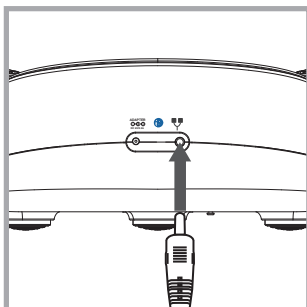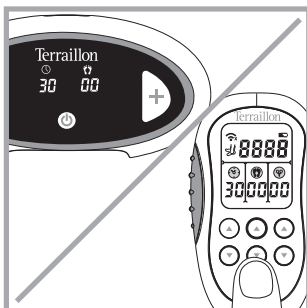
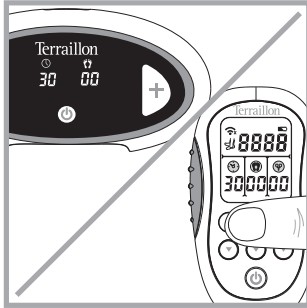
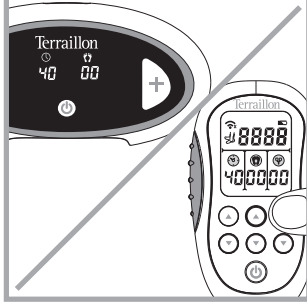
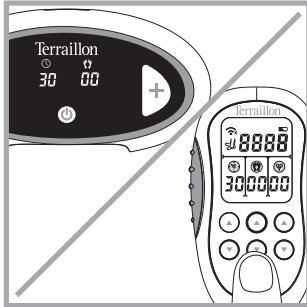
## POWER STEPPER - INSTRUCTION MANUAL

	<p><b>Step 5 - Start treatment and adjust intensity</b></p> <ul style="list-style-type: none"> <li>Using either the remote controller or buttons on the device to set the intensity level ▲/▼ [👣]. The treatment will begin and the [🕒] display start counting down the remaining minutes of treatment.</li> </ul> <p><b>CAUTION:</b></p> <ul style="list-style-type: none"> <li>We recommend using a heel balm on your feet to improve conductivity and stimulation.</li> <li>Each person may require a different intensity level to obtain the optimal muscle stimulation. The device can safely be used on intensity level 1 to maximum level 99.</li> <li>There are 15 therapeutic modes working as a cycle. 15 therapeutic modes will work alternately, every 1 minute it will change a mode.</li> <li>When use POWER STEPPER, the footboard will effortlessly tilt back and forth as per your feet rocking motion.</li> <li>If you remove your foot from foot pads, the device will give three short “Beep” sound (if intensity &gt; 10). Intensity of stimulation will come down to 10.</li> <li>Once put your feet back to the foot pads, the intensity level will be increased slowly to the level you set.</li> <li>During this period, the display shows the intensity level you set and no change immediately.</li> </ul>
	<p><b>Step 6 - Turn OFF the device</b></p> <ul style="list-style-type: none"> <li>Press the power button [🔌] on the device or hold 2s the ON/OFF button on remote control.</li> <li>If the device stop treatment (Intensity = 0), the device will be turned off automatically after 2 minutes.</li> <li>If the remote control is disconnected with the device, the remote control will be turned off automatically after 1 minute.</li> </ul>

## BODY STIMULATION (👣)

	<p><b>Step 1 - Attach pads to the cables</b></p> <ul style="list-style-type: none"> <li>Take the pads out of the sealed package.</li> <li>Insert the clip of the cable to the electrode pads.</li> <li>Make sure the end caps are fully inserted to the end.</li> </ul>
	<p><b>Step 2 – Connect the electrodes to the device.</b></p> <ul style="list-style-type: none"> <li>Insert the other end of the electrode power cable into the socket on the back of Power Stepper.</li> </ul> <p><b>CAUTION:</b> Ensure that your device is turned OFF.</p>

## POWER STEPPER - INSTRUCTION MANUAL

	<p><b>Step 3 - Pad placement</b></p> <ul style="list-style-type: none"> <li>Remove the clear plastic film from the back of the pad.</li> <li>Place pads on clean, dry and healthy skin near or surrounding the target specific body areas in order to relieve local pains and/or stimulate precise muscles:</li> </ul> <p>*Legs, thigh, knees *Shoulders, back, arms *Hips, waist</p>
	<p><b>Step 4 - Turn on the device</b></p> <ul style="list-style-type: none"> <li>Press the power button [⏻] on the device or use the remote control.</li> </ul> <p>In the default setting.</p> <ul style="list-style-type: none"> <li>The screen displays [🕒] 30 minutes, [👣] 00 output intensity, [👣] 00 output intensity. [👣] display 0000.</li> </ul>
	<p><b>Step 5- Setting the treatment time</b></p> <ul style="list-style-type: none"> <li>Set the treatment time by using the ▲/▼ [🕒] buttons on the remote control.</li> <li>The treatment time can be adjusted from 5 to 90 minutes.</li> </ul>
	<p><b>Step 6 - Start treatment and adjust intensity</b></p> <ul style="list-style-type: none"> <li>Set the intensity level by using the ▲/▼ [👣] buttons on the remote control.</li> <li>The treatment will begin and the [🕒] display will start counting down the remaining minutes of treatment.</li> </ul> <p><b>CAUTION:</b></p> <ul style="list-style-type: none"> <li>Each person may require a different intensity level to obtain the optimal muscle stimulation.</li> <li>The device can safely be used on intensity level 1 to maximum level 99.</li> <li>If the stimulation levels are uncomfortable or become uncomfortable, reduce the stimulation intensity to a comfortable level and contact your medical practitioner if problems persist.</li> <li>For safety purposes, this device was designed with load detection. If the electrode pads are removed from the skin, an alarm buzzer will sound and the intensity value will return to zero automatically.</li> </ul>
	<p><b>Step 7 - Turn OFF the device</b></p> <ul style="list-style-type: none"> <li>Press the power button [⏻] on the device or hold 2s the ON/OFF button on remote control.</li> <li>If the device stop treatment (Intensity = 0),the device will be turned off automatically after 2 minutes.</li> <li>If the remote control is disconnected with the device, the remote control will be turned off automatically after 1 minutes.</li> </ul> <p><b>For more information on the use of electrodes, please visit <a href="http://www.terraillon.com">www.terraillon.com</a></b></p>

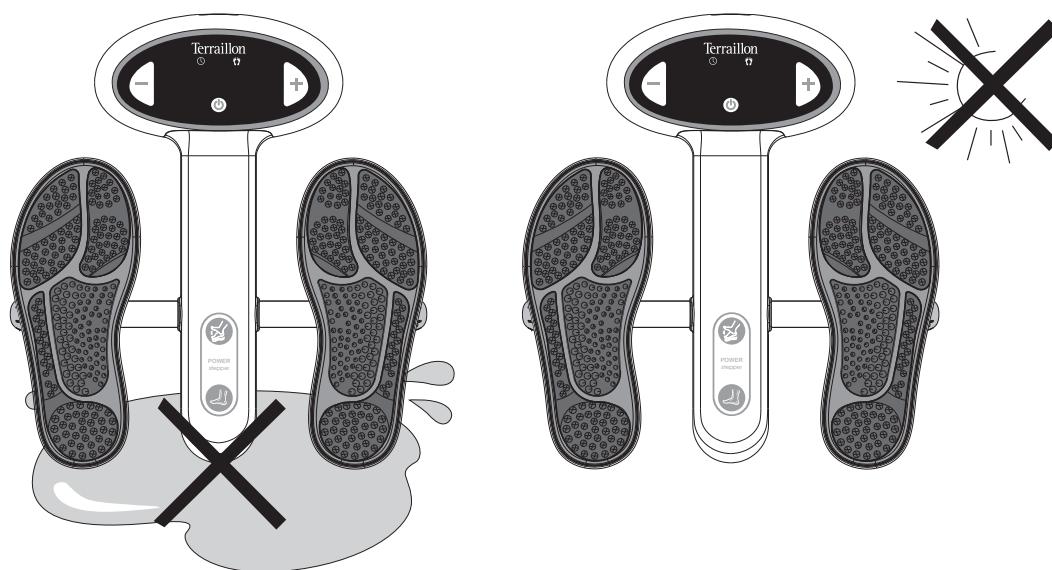
## POWER STEPPER - INSTRUCTION MANUAL

**SPECIFICATIONS**

- Channel: 3 channels
- Power Supply:  
Adaptor: input 100V- 240V ~, 50/60Hz; output: DC6V, 300mA;  
Or DC 6.0V, 4\*C battery (optional)
- Waveform: Bi-phasic waveform
- Frequency: 10Hz~36Hz
- Pulse Duration: 250µs for foot stimulation; 200µs for body stimulation
- Output Voltage: 0~80Vpp (at 500 ohm load)
- Treatment Time: 5~90 min
- Operating Conditions: 5°C to 40°C; 30%RH to 80%RH; 700hPa-1060hPa
- Storage Conditions: -10°C to 55°C; 10%RH to 90%RH; 700hPa-1060hPa
- Size: 416 x 380 x 104mm
- Weight: About 2100g (main device only, without batteries)
- Applied part: electrode pads and footpads
- Service life of the device: 3 years

**MAINTENANCE**

- Do not immerse device in water or any liquid. Do not drop the device or throw it from a height.
- Pull out the mains plug of the mains adaptor from the wall socket
- Remove from towel and allow to air dry at room temperature away from heat and sunlight.

**CLEANING****CLEANING THE UNIT**

- Turn unit off and disconnect the lead wires from the unit.
- Clean with a lightly moistened cloth (or a cloth soaked in a neutral cleaning solution) and wipe gently.
- Do not use chemicals (like thinner, benzene).
- Do not let water get into the internal area.

**CLEANING THE ELECTRODE PADS**

- Turn the power off and remove the lead wires from the pads.
- Wash the pads when the adhesive surface becomes dirty and/or the pads are difficult to attach.

## POWER STEPPER - INSTRUCTION MANUAL

- Wash the pad softly with your fingertips under slow running cold water several seconds (do not use a sponge/cloth/sharp object like a nail on adhesive side, do not use detergents, chemicals or soap).
- Dry the pads and let the adhesive surface air-dry completely (do not wipe with a tissue paper or cloth).

### CAUTION:

- The life of pads may vary by the frequency of washing, skin condition, and storage state.
- If the pad no longer sticks to your skin or the pad is broken, you should replace new pads.
- Before applying the self-adhesive electrodes, it is recommended to wash the skin, and then dry it.
- Do not turn on the device when the electrodes are not positioned on the body.
- Never remove the self-adhesive electrodes from the skin while the device is still turned on.
- If replacement electrodes are necessary, use only electrodes that are the same size (40\*40mm) as the electrodes provided with the device.
- Use of electrodes that are larger may reduce the effect of the stimulation. Use of electrodes that are much smaller than the electrodes provided with the device may increase.

## STORAGE

- Remove the batteries from the unit if you are not going to use it for a long period of time. Leaking batteries can damage the unit.
- Do not expose the device to direct sunlight and protect it against dirt and moisture.
- Store the device in a cool, well-ventilated place.
- Never place any heavy objects on the device.

## TROUBLESHOOTING

Problem	Check Points	Possible solution
The unit does not turn on.	Are the batteries exhausted?	Replace the batteries.
	Are the batteries installed correctly?	Insert the batteries observing polarity.
	Is the adaptor connected to the device and main socket correctly?	Re-connect the adaptor to the device and main socket according to “ Steps to connect the Power stepper ” in this manual.
The remote controller cannot power on	Is the battery exhausted?	Replace the battery.
	Is the battery install correctly?	Insert the battery observing polarity.
Device is on, but cannot feel the electrical stimulation	You feet may be dry.	Moisturize the sole of your feet preferably with a heel balm high in urea from your local pharmacy and try the procedure again. You may also have to increase the intensity level.
While putting your feet back to foot pads, the intensity won't turn back to the preset level.	Placing both bare feet on the foot pads at the same time.	Ensure that your feet are bare and each foot is placed on each of the foot pads at the same time. Keep increasing the intensity up to a maximum of 80 until you can feel the stimulation.
	You may be dehydrated.	Drink plenty of water before and after using the device. The device uses your body to create the electrical circuit. Water is an excellent conductor of electricity and if your body is less hydrated (below 60%) then the stimulation may be less. Therefore it is important to always remain well hydrated.
Output Stimulation is weak.	Is the intensity too weak?	Use a higher intensity level.

## POWER STEPPER - INSTRUCTION MANUAL

Stimulation is uncomfortable	Is the intensity too high?	Choose a lower intensity.
	Electrode active area size is too small.	Replace electrodes with ones that have an active area no less than 16.0 cm <sup>2</sup> (4 cm x 4 cm).
	Are you using the device correctly ?	Please check manual before the use.
My legs are aching after treatment.	You may have the intensity on a too high setting and your muscles are working too hard.	Choose a lower intensity and leave adequate time after each treatment to allow the muscles to recover (just like after vigorous exercise!).
Output current stops during therapy	Is the adaptor properly connected with unit?	Firmly connect the adaptor with the unit.
	Have the batteries been exhausted?	Please replace them with new batteries.

## NORMALIZED SYMBOLS

Caution!

Applied part of type BF.

Type of protection against electric shock: Class II Equipment.

Disposal in accordance with Directive 2012/19/EU (WEEE).

The name and the address of the manufacturer.

Date of manufacture.

Complies with the European Medical Device Directive (93/42/EEC) and amended by directive 2007/47/EC requirements. Notified body TÜV Rheinland (CE0197).

Authorized representative in the European Community.

Refer to Instruction Manual.

Keep dry.

Serial number.

IP21

The first number 2: Protected against solid foreign objects of 12,5 mm and greater. The second number: Protected against vertically falling water drops. Vertically falling drops shall have no harmful.

## POWER STEPPER - INSTRUCTION MANUAL

**PROTECT THE ENVIRONMENT**

Used batteries should be placed in the appropriate collection bin so they can be recycled. Do not mix different types of batteries, or use new and old batteries at the same time. At the end of its serviceable life, this product must be disposed of at a recycling centre for electrical and electronic equipment.

**WARRANTY**

This product is guaranteed against defects in materials or manufacturing. During the guarantee period, any such defects will be replaced free of charge (proof of purchase must be shown in the case of a complaint under guarantee). The guarantee does not cover damage resulting from improper use or negligence. In the event of a claim, first contact the shop where you purchased the product.

## POWER STEPPER - INSTRUCTION MANUAL

**ELECTROMAGNETIC COMPATIBILITY (EMC)**

- Use of accessories other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
- When the operating environment is relatively dry, strong electromagnetic interference usually occurs. At this time, the device may be affected as follows:
  - the device stops output;
  - the device turns off;
  - the device restarts;
 The above phenomenon does not affect the basic safety and essential performance of the device, and the user can use it according to the instruction. If you want to avoid the above phenomenon, please use it according to the environment specified in the manual.

Table 1

Declaration - electromagnetic emission		
The device is intended for use in the electromagnetic environment specified below. The customer or the user of device should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The device is suitable for domestic establishment and in establishment directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

Table 2

Declaration - electromagnetic immunity			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of device should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	± 2 kV for power supply lines ± 1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 0.5kV, ± 1 kV line(s) to lines ± 0.5kV, ± 1 kV, ± 2 kV line(s) to earth	± 0.5kV, ± 1 kV line(s) to lines ± 0.5kV, ± 1 kV, ± 2 kV line(s) to earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0 % UT; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % UT; 1 cycle and 70 % UT; 25/30 cycles Single phase: at 0° 0 % UT; 250/300 cycles	0 % UT; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % UT; 1 cycle and 70 % UT; 25/30 cycles Single phase: at 0° 0 % UT; 250/300 cycles	Mains power quality should be that of a typical commercial or hospital environment. If the user of the device requires continued operation during power mains interruptions, it is recommended that the device be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE: UT is the a.c. mains voltage prior to application of the test level.			

## POWER STEPPER - INSTRUCTION MANUAL

Table 3

Declaration - electromagnetic immunity			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of device should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3 V 0.15 MHz to 80 MHz 6 V in ISM and amateur radio bands between 0.15 MHz and 80 MHz	3 V 0.15 MHz to 80 MHz 6 V in ISM and amateur radio bands between 0.15 MHz and 80 MHz	Portable and mobile RF communications equipment should be used no closer to any part of device, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = 1.2 \sqrt{P}$ 150 kHz to 80 MHz $d = 1.2 \sqrt{P}$ 80 MHz to 800 MHz $d = 2.3 \sqrt{P}$ 80 MHz to 2,7 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, <sup>a</sup> should be less than the compliance level in each frequency range. <sup>b</sup> Interference may occur in the vicinity of equipment marked with the following symbol: <div style="text-align: center;">  </div>
Perturbations RF rayonnées CEI 61000-4-3	10 V/m 80 MHz à 2,7 GHz	10 V/m	
NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.			
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
a Field strengths from fixed RF transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which device is used exceeds the applicable RF compliance level above, device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating device.			
b Over the frequency range 0.15 MHz to 80 MHz, field strengths should be less than 3 V/m.			

Table 4

Recommended separation distances between portable and mobile RF communications equipment and device			
The device is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and device, as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	0.15 MHz to 80 MHz $d = 1.2 \sqrt{P}$	80 MHz to 800 MHz $d = 1.2 \sqrt{P}$	80 MHz to 2,7 GHz $d = 2.3 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer			
NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.			
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

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**Terraillon SAS France & Headquarters**
**Terraillon UK Limited**
**Terraillon Asia Pacific Ltd**
**Terraillon Corp USA**