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Dear Customer,

Thank you for purchasing this product. To be completely satisfied with your product, we recommend reading this user manual carefully.

If you have any difficulties or need more information, you can find our help centre by scanning the QR code opposite with your smartphone. If you cannot find the answer to your question, you can also contact our After Sales Service: support.uk@terraillon.com

PRINCIPLE OF OPERATION: TENS & EMS

The NECK CARE+ with its TENS + EMS dual band output is effectively relieving neck pain, muscle fatigue and stiffness.

1) What is Transcutaneous Electrical Nerve Stimulation (TENS)?

TENS (Transcutaneous Electrical Nerve Stimulation) is now widely recommended by doctors, physiotherapists, and pharmacists around the world. It is a non-invasive, drug-free method for managing pain. TENS works by sending small electrical impulses through the skin to the nerves, which helps to alter the way pain is perceived. It is important to note that TENS does not cure any underlying condition—it only helps to control pain. Results may vary from person to person, but for most users, TENS can reduce or eliminate pain, making it easier to return to normal daily activities.

Working principle and benefits:

Scientific studies suggest that electrical stimulation therapy may help relieve pain in a few ways. The gentle electrical pulses pass through the skin to the nearby nerves, blocking pain signals from reaching the brain. At the same time, these pulses can stimulate the body to produce more endorphins, which are natural substances that help reduce pain.

2. What is Electrical Muscle Stimulation (EMS)?

EMS works by sending gentle electrical pulses to the muscles that need treatment. These pulses cause the muscles to contract and relax, creating a form of passive exercise. The stimulation is produced using a square wave pattern, which allows the impulses to act directly on the muscle motor nerves.

Working principle and benefits:

EMS units send gentle, comfortable electrical impulses through the skin to stimulate the nerves in the targeted muscles. When a muscle receives these signals, it contracts as if the brain had sent the command itself. As the signal strength increases, the muscle flexes, similar to normal exercise, and then relaxes when the pulse stops. This cycle is repeated to create muscle contractions or vibrations. Normally, muscle movement is controlled by the nervous system, which sends electrical signals from the brain to the muscles. EMS works in a similar way but uses an external device with electrodes on the skin to deliver the impulses. The nerves transmit the signal to the targeted muscle, which reacts by contracting, just like it does during natural muscle activity.

By combining both technologies and using different pulse frequencies, this device can help relieve neck pain through the various programs available.

CONTRAINDICATIONS

The patient is an intended operator. Consult with your physician before, because the device may cause lethal rhythm disturbances in certain susceptible individuals.

DO NOT USE THIS DEVICE UNDER THESE CONDITIONS:

- If you have a cardiac pacemaker, implanted defibrillator, or other implanted metallic or electronic device. Such use could cause electric shock, burns, electrical interference, or death.
- Do not use the therapeutic device together with a high-frequency (HF) apparatus, as this may cause burns or damage the device.
- Together with a life-supporting medical electronic device such as an artificial heart, lung or respirator.
- In the presence of electronic monitoring equipment (e.g., cardiac monitors, ECG alarms), which may not operate properly when the electrical stimulation device is in use.
- Patients with hemorrhage: Use is contraindicated during the acute/unsteady phase. For individuals with residual effects or sequelae, the device should be used only under medical supervision.
- It is contraindicated for use by those who have acute cardiovascular and cerebrovascular diseases.

- On open wounds or rashes, over swollen, red, infected, inflamed areas, or skin eruptions (e.g., phlebitis, thrombophlebitis, varicose veins); or on top of, or in proximity to, cancerous lesions.
- It is contraindicated for those who have a skin perceptual disturbance or cannot feel the heat.
- Metal allergies are contraindicated.
- On the opposite sides of your neck.

DO NO USE ON THESE INDIVIDUALS:

- Pregnant women, because the safety of electrical stimulation during pregnancy has not been established.
- Children or infants, because the device has not been evaluated for pediatric use.
- Persons incapable of expressing their thoughts or intentions.

DO NOT USE DURING THESE ACTIVITIES:

- Bathing or showering.
- Sleeping.
- Driving, operating machinery or any activity in which electrical stimulation can put you at risk for injury.

PRECAUTIONS

The following precautions should always be taken before using the NECK CARE+. Read the operating instruction carefully and keep them for future reference. If you pass this device on to others, please also pass on this manual.

GENERAL INFORMATION

- The patient should be the intended operator.
- The long-term effects of electrical stimulation are unknown.
- Apply stimulation to only normal, intact, clean, dry, and healthy skin.
- TENS is not effective in treating the original source or cause of the pain, including headache.
- TENS is not a substitute for pain medications and other pain management therapies.
- TENS devices do not cure disease or injuries.

- TENS is a symptomatic treatment and, as such, suppresses the sensation of pain that would otherwise serve as a protective mechanism.
- Effectiveness is highly dependent upon patient selection by a practitioner qualified in the management of pain patients.
- Do not connect the patient simultaneously to the stimulator and any high-frequency surgical medical electrical (ME) equipment, as this may cause burns at the electrode sites and potential damage to the stimulator.
- Avoid operating the stimulator within close proximity (e.g., within 1 meter) of short wave or microwave therapy ME equipment, as this may cause instability in the stimulator output.
- You may experience skin irritation or hypersensitivity due to electrical stimulation from the electrodes.
- If you have suspected or diagnosed heart disease, you should follow precautions recommended by your physician.
- If you have suspected or diagnosed epilepsy, you should follow precautions recommended by your physician.
- Use caution if you have a tendency to bleed internally, such as following an injury or fracture.
- Consult with your physician prior to using the device after a recent surgical procedure, because stimulation may disrupt the healing process.
- This stimulation should not be applied over areas of skin that lack normal sensation.
- Keep unit away from young children. The unit contains small pieces that may be swallowed. Contact your physician immediately if ingested. The device contains button batteries and small parts, please put this product out of reach of young children to prevent them from eating and swallowing it by mistake.
- This product contains connecting wires. Please prevent children from misusing them to avoid strangulation accidents.
- Do not modify the device or the remote without authorization of the manufacturer. This could cause improper functioning. Do not use the device remote if it is damaged. Always check the device and the remote for damage before use.
- Please do not maintain or repair the equipment during use, or it may cause danger.

- Please do not disassemble or reassemble the product, except when it needs to be disposed of, in accordance with the instructions provided later in this manual.
- When need replacing the lithium batteries, please contact after sales staff designated and authorized by the manufacturer directly, its accessories shall not be replaceable.
- Product battery replacement should be replaced by professional maintenance staff, otherwise it will produce risk.
- Please pay attention to the positive and negative polarity when replacing the button battery, or it may damage the device.
- Dust and lint may affect the performance of the unit, please use a dry soft cloth to clean the device as needed.

PAIN MANAGEMENT WARNINGS

- If you have had medical or physical treatment for your pain, consult with your physician before using this device.
- If your pain does not improve, becomes seriously chronic or severe, or continues for more than five days, stop using the device and consult with your physician.
- The mere existence of pain functions as a very important warning telling us that something is wrong. Therefore, if you suffer from any serious illness, consult your physician in order to confirm that it is advisable for you to use this unit.
- The pulse intensity should be adjusted to a comfortable intensity that is acceptable. The increase of the intensity will not necessarily enhance the comfort of the massage and the high level of intensity may be strongly stimulating.

USAGE PRECAUTIONS

- If the unit is not functioning properly or you feel discomfort, immediately stop using the device.
- Do not use for any other purpose except as described in this manual.
- Please pull up the power plug when you are done or do not use the product.
- The therapeutic apparatus should be placed in the position which is easy to move the plug to use, it is easy to move the plug in emergency circumstances.
- The long-term effects of chronic electrical stimulation are unknown, please do not use longer than 30 minutes every time.

- Only use remote approved by the manufacturer. The use of unapproved remote may damage the device or injure users.

DO NOT USE THE DEVICE THIS WAY:

- The product must not be moved or changed in position during use, it should be switched off, otherwise there will be strong irritation.
- Do not place the electrodes under any circumstances on the head, skull, scalp, eyes, mouth, or the front of the neck (especially the carotid sinus).
- The electrodes of the neck should not touch any metal object such as jewelry made from metal.
- Do not disassemble the product yourself, and prevent collisions and water ingress, otherwise malfunctions will be caused.

POSSIBLE ADVERSE REACTIONS:

- Do not use device to treat one region for extended periods of time (15 minutes per session, maximum 2 times per day) or muscles in that region may become exhausted and sore.
- You may experience skin irritation and burns beneath the stimulation electrodes applied to your skin.
- You should stop using the device and consult with your physician if you experience adverse reactions from using the device.

 **This device may only be used for the intended purpose described in this manual. The manufacturer cannot be held liable for incidental, consequential or special damage caused by misuse or abuse.**

PACKAGE CONTENTS

Neck Massage
Device
x1

Remote
control
x1

Instruction
manual
x1



CR2032
battery
x1

USB-C
cable
x1

PRODUCT DIAGRAM

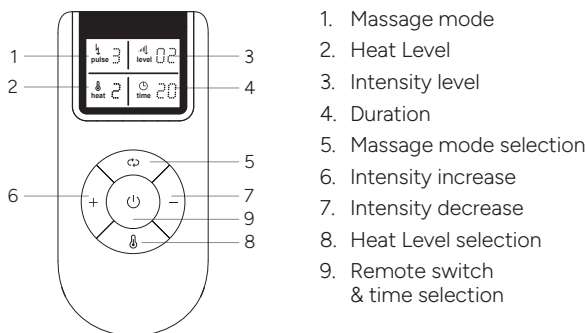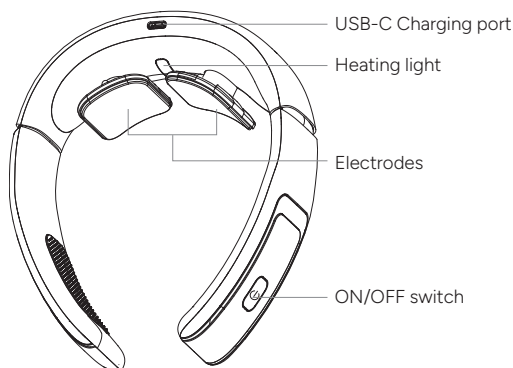PRODUCT INTRODUCTION

The NECK CARE+ is designed based on traditional Chinese medicine principles, focusing on nourishing qi and blood. It helps promote blood circulation, relieve muscle discomfort, reduce pain, and improve meridian flow.

Combining long-term clinical experience with modern electronic technology, this high-tech device is tailored for neck care and prevention. Its streamlined ring shape fits the natural curvature of the neck, ensuring comfort during use.

The device uses LF electrotherapy to generate an effective composite energy field that penetrates deep tissues, supporting treatment of cervical vertebra issues. Throughout the treatment, the device maintains proper physiological positioning, providing a safe and scientific approach.

The impedance technology provides an adaptive experience for every neck, automatically adjusting the stimulation intensity according to your individual skin moisture.

1. PRODUCT INTENDED USE

- Intended use: promotes neck blood circulation and relieve pain. Design for contact with the skin of the neck. Recommended use Time: 15-30min per day.
- Intended user: adults aged between 18 and 60 years old who experience muscle fatigue and neck pain and need to improve blood circulation.
- Benefits: relieves pains, promotes blood circulation, effectively reduces The Neck Disability Index (NDI), enhance cervical range of motion (ROM).


WARNING:

In case of any adverse events or serious complaints in the EU market, please contact EC Representative and the manufacturer as soon as possible, so that the manufacturer can confirm, handle and reply in time.

2. DEVICE FEATURES

Pulse functions

There are two automatic combinational massage modes and four manual massage modes to be fit for wider range of people, and it can be selected according to different needs.

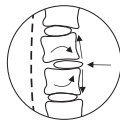The bionic current skin impedance detection technology, allows automatic adjustment of pulse output, and giving a more comfortable experience.

Conducted through the skin by metal patches, it acts directly on the sensory nerves, exercises the neck muscular tissue and relieves neck discomfort.

Far infrared hot compress function

Far infrared heating (8–14 μm) provides gentle warmth to the neck area, promoting blood flow and relaxation.

Red light irradiation function

Red light (630–650 nm) acts on the skin through light-based stimulation, enhancing comfort and relieving tension.

Circumferential traction

With its 3D intelligent adjustment technology and a 15°–30° tilt, it naturally fits the curve of the neck, providing a comfortable, personalized, and aesthetically pleasing experience.

WARNING:

Do not place on any other parts of the body, not indicated in this instruction manual.

PREPARATION BEFORE USE

1. POWERING UP THE PRODUCT

Before the first use, charge the device to maximum capacity.

- Remote control: insert the CR2032 battery.
- Main device: connect cable to device via the USB-C port and connect the other end to a USB-C adapter.

When charging, the main device it is blinking in orange. When battery is fully charged (about 2 hours), the light becomes steady. USB cable can be disconnected and device operated.

Fully charged device can last about 5 days with 30 minutes usage per day.

It will emit a buzzing sound three times when the battery is not sufficiently charged.

Notes:

If the rechargeable battery is not used for an extended period, their service life may be reduced. It is recommended to fully charge the battery at least once per month. If the product will not be used for an extended period, remove the button battery to prevent battery leakage, which may cause malfunction or damage to the device.

2. SET UP

Wear the massager on the neck and have the electrodes against the skin of the back of the neck.

Notes:

- If there're sweat stains on the cervical area, clean it before use.
- When the massager detects dry skin, the massager will limit the intensity of the pulse massage and give an alert (8 continuous beeps) until the skin is not dry, to avoid stinging.

3. START-UP AND MODE CHOICE

1. Once the device in place, press the  button on the right handle to switch on the massager, and the massager then enters the standby state and the working light bar flows in a cycle from left to right and then from right to left.
2. Press the  key on the remote control to switch the device on. The default program starts in the following mode: pulse mode «1», intensity at 0, Heat is set at level 2 (medium) with the duration set at 15 min. You can operate the remote control to adjust the desired massage technique, intensity, heat level and duration ; when the main unit successfully receives a signal by operating the remote control, a beep will sound.

Massaging mode set up

Modes 1-6 are available by pressing the  pulse mode key.

The 6 massaging modes are as follow:

Massage modes 1 to 5 use a combination of EMS and TENS at low and medium frequencies, mode 6 is using EMS only at low frequency.

	Mode	Functions	Benefits	Users
1	 Auto-Cycle 1	Automatically cycles through scart, massage & malaxation modes.	Full muscle stimulation and relaxation.	Perfect if you are not looking for a specific massage mode. Perfect for an easy going relaxing massage.
2	 Auto-Cycle 2	Automatically cycles through scart, massage & thumb modes.	Alternating stimulation for both relaxation & targeted relief.	Perfect if you are not looking for a specific massage mode. Easy option to relieve overall stiffness.
3	 Scart	Delivers short, continuous, sharp percussions along the muscle.	Stimulates muscles, help improves blood flow, relieves fatigue.	Users needing quick muscle stimulation or energizing massage.
4	 Massage	Intermittent percussion simulating gentle massage strokes.	Provides gentle massage, relaxes muscles, eases tension, supports fatigue relief.	Ideal for a daily relaxation, light muscle relief.
5	 Malaxation	Alternates slow, deep pulsing patterns simulating kneading effect.	Deep muscle relaxation and relieves knots, improves flexibility.	Ideal to target knots, overall tension and deep muscle stiffness.
6	 Thumbs	Delivers rhythmic percussive pulses that mimic tapping or thumping motions.	Targeted relief of muscle knots and tension points.	Users with localized tension or tight spots needing focused massage.

Intensity set up

Press the “+/-” button to switch the intensity of electric pulse, there are 16 gears intensity of electric pulse. Increase the intensity gradually. Thanks to impedance technology, the device automatically adapts to your skin, which may take a few seconds before the massage is fully perceptible. Once the adaptation is complete, select the intensity level that feels most comfortable for you.

Heating mode set up

Press the  button, to switch the heating mode, there is 3 heat level, low, medium & high.

Time set up

Press the button  on the remote, to switch the timing. It goes from 5 to 30 minutes, and increase by 5 minutes once pressing the button .

If you want to stop using, you maintain the button  press on the handle or the remote until the light goes out to switch off the device.

PRECAUTIONS

It is suggested to use the therapeutic device twice a day, with electric pulses for 10–15 minutes each time, and heat therapy for 15–30 minutes each time.

MAINTENANCE & CARE

CLEANING


- To clean the product, wipe with a soft cloth or towel with a little water; do not use volatile agents.
- After each use, please clean conducting plate, you can wipe by using soft cloth or towel with a little water and after dry. When conducting plate is particularly dirty, you can wipe by using soft cloth with a little medicinal alcohol (alcohol concentration as 75%).
- Please store it in a dry and well-ventilated place.
- Do not store it in direct sunlight, hot or humid places.

DISPOSAL


To prevent environmental pollution, do not discard the NECK CARE + or its accessories with household waste. When they can no longer be used, remove the battery and dispose of them according to local environmental regulations.

- The built-in lithium batteries contains substances that may contaminate the environment. Before disposal of it, remove the built-in lithium batteries.
- Always switch off or disconnect the power before disassembling the massager and removing the lithium batteries.
- If the device is damaged or shows any leakage, do not attempt to open it. Please contact the company after purchase for assistance.
- To remove the lithium battery, proceed as shown in the diagram below, and disassemble the product as per the steps from 1 to 3.


1. Open the housing.
Use a tool such as a flat screwdriver to act as a pendulum.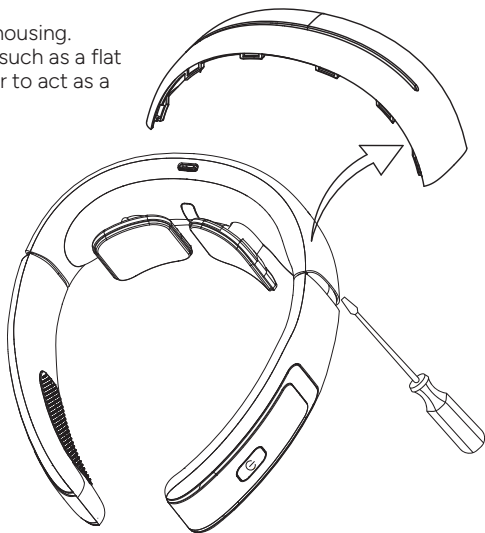

2. Take out the accessories.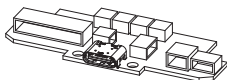

3. Take out the battery.

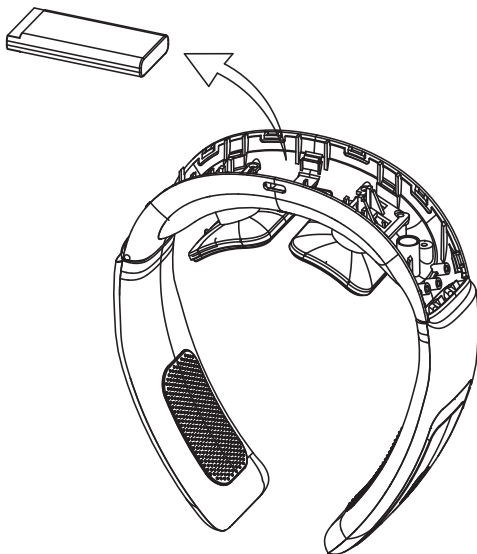WARRANTY AND ENVIRONMENTAL PROTECTION

This appliance is guaranteed against material and manufacturing defects for two years. During this period, the product will be replaced free of charge (proof of purchase must be presented if the product is under warranty). This warranty does not cover damage resulting from accidents, improper use or negligence. If you have a complaint, please contact the store where you purchased your product first.

Waste electrical products should not be disposed of with household waste.

Please recycle where facilities exist. Check with your local authority or retailer for recycling advice.

TECHNICAL SPECIFICATIONS

Model	P7
Display	LCD screen
Pulse	Impulse frequency: 1Hz~400Hz. Pulse modulation frequency: 4KHz-10KHz Pulse width: 20-400μs Pulse waveform: Square wave Individual pulsing electric quantity when in the maximum output amplitude: >7μC. Maximum output energy of individual pulse: ≤300mJ. Effective value of maximum output amplitude: ≤25V (50mA) Output voltage peak, in open measurement: ≤500V. Influence of output end open circuit and short circuit: Supports open and short circuit at the output end; performance will not weaken Adjustment of output amplitude: Continuous and balanced; minimum output ≥ 2% of maximum output
Rated load impedance	500 Ω ± 10%
Heating	Hot compress temperature: +38°~48°C (at an ambient temperature of 23°C) Red light wavelength: 630~650 nm
DC component	None
Dimensions	Box dimensions: 171 x 61 x 213 mm Main device dimensions: 155 x 139 x 43 mm Remote dimensions: 91 x 42 x 12 mm
Weight	Approx. 213 g
Power source	Main device DC 5V1000mA power supply Lithium battery specification: 3.7V DC.1500mAh Battery life: around 500 times charge and discharge, about 3 years Remote: CR2032 3V DC of Button battery Battery life: 60 days (30 min/day) Charging time: approx. 2h

Degree of protection against electric shock	Type BF
Protection against ingress of water particles	IP22
Operating environment	Temperature: 5°C ~ 40°C Humidity: 15% ~ 93% RH Air pressure: 70 kPa ~ 106 kPa Under normal operating conditions (ambient temperature 20°C), the equipment requires 2 hours to reach a fully operational state after being stored at either its minimum or maximum storage temperature.
Storage conditions	Store in a clean and safe place. Storage temperature: -25°C ~ 55°C Humidity: 0% ~ 93% RH Air pressure: 70 kPa ~ 106 kPa Store in a well ventilated area, away from corrosive gases.
Transport conditions	Transport temperature: -10°C ~ 40°C Humidity: 15% ~ 93% RH Air pressure: 70 kPa ~ 106 kPa
Standard	The product has undergone clinical testing in compliance with ISO 13485

Declaration: The P7 electric neck massager has been tested in accordance with the recommendations of technical report IEC 60601-1:2005+A1:2012 Medical electrical equipment

Part 1: General requirements for basic safety and essential performance. EN60601-1-11:2015 Medical electrical equipment General requirements for basic safety and essential performance- Collateral standard: Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment. EN60601-2-10 (2nd edition) Medical electrical equipment

Part 2-10: Particular requirements for the safety o

Part 1-2: General requirements for safety — Collateral standard: Electromagnetic compatibility — Requirements and tests.

Hereby, Terrailon SAS declares that the product Neck Care+ complies with the essential health and safety regulations. This product meets the fundamental requirements and other applicable provisions of the European RED Directive 2014/53/EU. The full declaration of conformity is available at: <https://terrailonhelp.zendesk.com/hc/en-us/categories/360000322733-Declaration-of-Conformity>
E.I.R.P: -10.3 dBm

TROUBLESHOOTING

PROBLEM	POSSIBLE CAUSES	SOLUTION
Cannot be switched on.	Is the massager in standby mode?	Press the right handle ON/OFF button to enter standby mode.
	Is the button batteries of remote control low?	If there's no display, please replace the button batteries CR2032 in the remote control with the new ones.
	Is the rechargeable battery inside the main unit sufficiently charged?	If the rechargeable battery is low, please charge it.
	Is it charging properly?	Check that the USB cable is plugged in place when charging; the massager charging LED lights up when the cable is correctly connected.
No charge or no charging display.	Is the USB cable properly connected?	Check that the USB cable is plugged in place.
	Is the input power supply used DC 5.0V?	Confirm the specification of the input power supply.
Normal display, but no response or no sensation.	Do the electrodes of the massager come into direct contact with the skin of the cervical spine?	The electrodes of the massager must be in direct contact with the skin of the cervical spine.
	Are there any objects such as hair, clothing or accessories in contact with the electrode and the skin?	Please remove or separate these items.
	Is the massage intensity adjusted?	The default state of the massager after switching on is the lowest intensity, and you need to adjust the intensity; some customers may also increase the intensity appropriately based on their needs.

PROBLEM	POSSIBLE CAUSES	SOLUTION
There's a tingling sensation.	Do the massager electrodes fit snugly against the skin of the neck?	Moisten the skin of the neck with a damp towel before applying the electrodes firmly to the skin and adjusting the appropriate intensity and massage mode.
	Does the massage take too long?	Suitable duration is 10 to 15 min per time, no more than twice a day.
	Is the massage too intense?	Turn down the intensity as appropriate.
	Is the skin allergic to electrodes?	In case of allergy, please consult your doctor for advice.
	Cold weather and dry skin.	Wet the skin and the electrode surface with a damp towel.
	Use after a late night.	Please use every other day after a late night due to a drop in electrolytes.
	Is there any hair or jewelry in contact with the skin at the electrodes?	Please remove or separate accessories and hair.
Low stimulation.	Is the skin of the neck pressed against the electrodes?	Press firmly against the skin and adjust to the appropriate intensity.
	Are there any dirt or oil stains on the electrodes?	Clean the electrode surface.
	Is the battery low?	If the battery is low, please charge it with an external power supply.
Red spots appeared on the skin after use.	Is there a strong stinging sensation when using it?	Please stop using the massager, and it will usually disappear in 3-5 days.

SAFETY INFORMATION

Knowledge of the relevant warning signs and symbols is essential for the correct and safe use of this medical device. The following signs and symbols may appear in this user manual or on the product labeling:

	Batch number
	Serial number
	Manufacturer
	Authorised representative in the european community
	Date of Manufacture
	Caution
	Type BF applied Part
	Medical device
	This product only for indoor use
	Keep out of direct sunlight
	Keep away from rain
	Electrical devices are recyclable material and should not be disposed of with household waste after their useful life. Help us protect the environment and save resources and by taking this device to the appropriate collection point. Please contact the organization which is responsible for waste disposal in your area if you have any questions.
	CE

IP22	The dustproofing and waterproofing grade, that is, to prevent the solid object is greater than 12mm to invade, the inclination of 15 degrees can prevent dripping invasion still not cause harmful effects.
	This symbol is used to direct the user to refer to documentation for additional information regarding the system use or description.
	Refer to instruction manual
	Importer in Switzerland
	Authorized representative in Switzerland

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EMC GUIDANCE

1. This product needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided, and this unit can be affected by portable and mobile RF communications equipment.
2. Caution: This unit has been thoroughly tested and inspected to assure proper performance and operation!
3. Caution: this machine should not be used adjacent to or stacked with other equipment and that if adjacent or stacked use is necessary, this machine should be observed to verify normal operation in the configuration in which it will be used.

Warning

The use of ACCESSORIES, transducers and cables other than those specified, with the exception of transducers and cables sold by the MANUFACTURER of the AMBULATORY Neck Electronic Muscle Stimulator as replacement parts for internal components, may result in increased EMISSIONS or decreased IMMUNITY of the Neck Electronic Muscle Stimulator.

Following cable types must be applied to ensure compliance with standard of radiation interference and noise immunity.

Cables	Length (m)	Whether shielded
Charging cable	0.5/1	No

Table 1

Guidance and manufacturer's declaration – electromagnetic emissions		
The MODEL P7 is compliance for each EMISSIONS test specified by the standard, e.g. EMISSIONS class and group.		
Emissions	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The P7 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emission CISPR 11	Class B	The P7 is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations / flicker emissions IEC 61000-3-3	Complies	

Table 2

Guidance and manufacturer's declaration – electromagnetic immunity		
The P7 is compliance for each IMMUNITY test specified by the standard, e.g. IMMUNITY test level.		
Immunity test	IEC 60601-1-2 test level	Compliance level
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air
Radiated RF EM fields IEC 61000-4-3	10V/m 80MHz-2.7GHz 80% AM at 1kHz	10V/m 80MHz-2.7GHz 80% AM at 1kHz
Electrical fast transient/ burst IEC 61000-4-4	±2 kV 100 kHz repetition frequency	±2 kV 100 kHz repetition frequency
Surge IEC 61000-4-5	±0.5 kV, ±1 kV line-to-line; ±0.5 kV, ±1 kV and ±2 kV line-to-ground	±0.5 kV, ±1 kV line-to-line; ±0.5 kV, ±1 kV, and ±2 kV line-to-ground;
Conducted disturbances induced by RF fields IEC 61000-4-6	3V 0.15MHz-80MHz 6V in ISM and amateur radio bands between 0.15MHz and 80MHz 80% AM at 1kHz	3V 0.15MHz-80MHz 6V in ISM and amateur radio bandsbetween 0.15MHz and 80MHz 80% AM at 1kHz
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0% UT : 0.5 cycle a) At 0o, 45o, 90o, 135o, 180o, 225o, 270o, and 315o.	0% UT : 0.5 cycle a) At 0o, 45o, 90o, 135o, 180o, 225o, 270o, and 315o.
	0% UT : 1 cycle 70% UT : 25/30 cycles b) Single phase: at 0o	0% UT : 1 cycle 70% UT : 25/30 cycles b) Single phase: at 0o
	0% UT : 250/300 cycles b)	0% UT : 250/300 cycles b)
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m 50 Hz or 60 Hz	30 A/m 50 Hz or 60 Hz
NOTE a) UT is the a.c. mains voltage prior to application of the test level; b) E.g. 25/30 means 25 periods at 50 Hz or 30 periods at 60 Hz.		

Table 3

Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment

Test Frequency (MHz)	Band (MHz)	Service a)	Modulation b)	Maximum Power (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)	Compliance level
385	380-390	TETRA 400	Pulse Modulation 18 Hz	1.8	0.3	27	27
450	430-470	GMRS 460 FRS 460	FM c) ± 5 kHz deviation 1 kHz sine	2	0.3	28	28
710	704-787	LTE-Band 13, 17	Pulse Modulation 217 Hz	0.2	0.3	9	9
745							
780							
810	800-960	GSM 800/900, TETRA 800, Iden 820, CDMA 850, LTE Band 5	Pulse Modulation 18 Hz	2	0.3	28	28
870							
930							
1720	1700 -1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse Modulation 217 Hz	2	0.3	28	28
1845							
1970							
2450	2400 -2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse Modulation 217 Hz	2	0.3	28	28
5240	5100 -5800	WLAN 802.11 a/n	Pulse Modulation 217 Hz	0.2	0.3	9	9
5500							
5785							

NOTE:

a) For some services, only the uplink frequencies are included.

b) The carrier shall be modulated using a 50 % duty cycle square wave signal.

c) As an alternative to FM modulation, 50 % pulse modulation at 18 Hz may be used because while it does not represent actual modulation, it would be worst case.

Terraillon SAS France & Headquarters

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EN Scan the QR Code to access the NECK CARE+ installation and operation video guide.

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Terraillon Asia Pacific Ltd